



## ML - G Medical X-Ray Film

- **MADE IN USA**  
GUARANTEED CONSISTENCY AND QUALITY
- **T-GRAIN EMULSION**  
HIGHER VISIBILITY OF ANATOMIC DETAIL.
- **SYSTEM VERSATILITY**  
USE WITH ANY SPEED GREEN RARE EARTH SCREEN.  
100: DETAIL SPEED: EXTREMITIES  
600: HIGH SPEED: HIGH DENSITY AREAS
- **ADVANCED TECHNOLOGY**  
REDUCED HANDLING AND  
PROCESSING ARTIFACTS
- **FULL SPEED**  
REDUCED RADIATION EXPOSURE
- **HIGH CONTRAST**  
ENHANCED IMAGE SHARPNESS

### PACKAGING

Available in all common sizes including metric. The easy open, reclosable box contains 100 sheets and is available in case quantities of 5 boxes per case.

### PROCESSING

ML-G's advanced emulsion technology is formulated to produce optimum results when processing in both automatic processors and manual developing systems. This film is designed to be processed in any brand of quality chemistry. For optimum results developer temperature in a 90 sec. processor should be approx. 95°F (35°C). Manual developing times will depend upon individual processing conditions and desired results.

### SAFELIGHT

A Kodak GBX-2\* or equivalent with a minimum 15 watts is recommended. The safelight should not be any closer than 4 feet from the loading area.

\*GBX-2 is a trademark of Eastman Kodak Co.

