



ML-HS Medical X-Ray Film

- **MADE IN USA**
GUARANTEED CONSISTENCY AND QUALITY
- **RARE EARTH SENSITIVITY**
DESIGNED EXCLUSIVELY FOR USE WITH BLUE
RARE EARTH INTENSIFYING SCREENS.
- **HIGH CONTRAST**
OPTIMUM DETAIL VISIBILITY
- **HALF SPEED**
REDUCES GRAININESS CAUSED BY HIGH SPEED
SCREENS .

PACKAGING

Available in all common sizes including metric. The easy open, reclosable box contains 100 sheets and is available in case quantities of 5 boxes per case.

PROCESSING

ML-HS's advanced emulsion technology is formulated to produce optimum results when processing in both automatic processors and manual developing systems. This film is designed to be processed in any brand of quality chemistry. For optimum results developer temperature in a 90 sec. processor should be approx. 95°F (35°C). Manual developing times will depend upon individual processing conditions and desired results.

SAFELIGHT

A Kodak GBX-2* or equivalent with a minimum 15 watts is recommended. The safelight should not be any closer than 4 feet from the loading area.

*GBX-2 is a trademark of Eastman Kodak Co.

